

## Instructions for KN-95 mask usage

1. You will receive 5 KN-95 masks for your individual use over the next 2 weeks. NOW MORE THAN EVER, proper mask usage is ESSENTIAL. **KN-95 masks should be used instead of conventional cloth or paper ear loop masks.** Only N95 masks are acceptable alternatives.
2. Please wear a different mask daily, using a new mask on the first 5 consecutive days. SARS-CoV-2 needs a host to survive. By allowing the virus time to die off, you can safely reuse the KN-95 masks we are providing one time. It might be useful to help you keep track by numbering them 1-5 as soon as you receive them.
3. Masks should be worn at all times when indoors anywhere in the campus.
4. Masks should fit snugly across the bridge of the nose, covering the nose and mouth at all times.
5. When not wearing a mask, store the masks in a dry location, preferably in a paper bag.
6. After you have used all 5 masks, please reuse each mask one more time, again using a different mask each day.
7. After using each mask twice (10 days), please discard the masks and return for another distribution of masks.